EN MEDIBLINK NASAL ASPIRATOR 2-in-1 M400

Instructions for use


Intention of use: Mediblink nasal aspirator enables the quick, efficient, simple and safe removal of mucus from the nose in babies and children.

Warning: Wet the nasal cavity with sterile water before each use of the aspirator. During this procedure, keep your body in an upright position. The product can be operated only by adults. Potential choking hazard. Keep all parts out of reach of children.

Notice: If any serious incident has occurred in relation to the device, it should be reported to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.


You can safely use the device up to 2,500 times.

Use with a vacuum device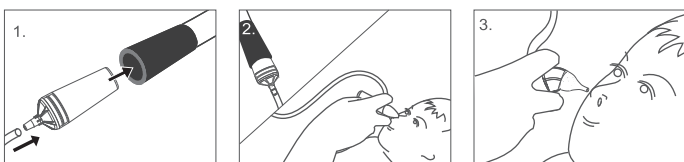

1. Assemble the device and insert the vacuum connector into the tube of the vacuum cleaner.
2. Place the baby on his/her back in a stable and comfortable position. Gently place the soft tip to baby's nostril.
3. Switch on the vacuum cleaner and adjust the necessary sucking force with the vacuum modulator of the vacuum cleaner.

Use by mouth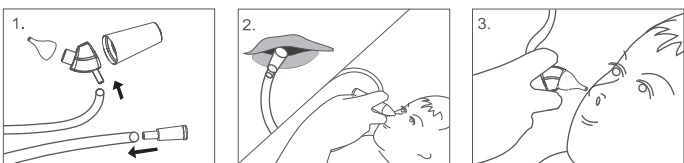


1. Connect mouthpiece and tube, place the baby on his/her back in a stable and comfortable position.
2. Place mouth piece in the mouth while holding baby with one hand and gently place the soft tip to baby's nostril, forming a seal.
3. Always begin by sucking gently and slowly until all loose mucus has been removed.

After use cleaning

Disassemble the aspirator after each use, and separately flush each part of the aspirator under running water. All parts **except the External tank (1) and Cap (3)** should be disinfected in boiling water (5 min at 100 °C). To disinfect External tank (1) and Cap (3), dip both parts in 70% Ethanol alcohol for 3 seconds. Allow all aspirator parts to air-dry thoroughly. Keep parts fitted and in a clean place. Do not use any detergents, chemicals or abrasive materials on it.

Part No.	Material	Disinfection in boiling water	Disinfection with 70% Ethanol alcohol
5 6 8	PP	✓	×
1 3	PS	×	✓
2 4-1 4-2 7	Silicone	✓	×

Components

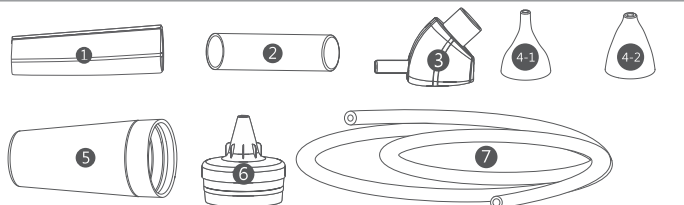- 1 External tank
- 2 Internal silicone tube
- 3 Cap
- 4-1 Soft silicone tip, narrow
- 4-2 Soft silicone tip, wide
- 5 Vacuum connector
- 6 Connection piece
- 7 Tube
- 8 Mouthpiece

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